


# Typical Cubicle Track MOUNTING DETAILS

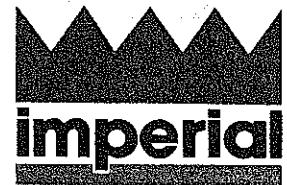

These details represent only a few of the possible methods for installing Imperial Fastener Cubicle and IV Track. If specific procedures are outlined in contract documents, verify with architect exactly which method to follow. Fastening devices noted

here are to be used as guidelines only. Specific sizes and spacing must be determined by installer. No fastening devices or any above-ceiling supports are furnished with track.

<p><b>Mounting Thru Suspended Ceiling to Furring Above</b></p> <p>#8 x 2" Tek Screws 24" O.C.</p> <p>Wired to Structure Above, As Required</p> <p>Continuous Metal Furring Over Track</p> <p>Ceiling Grid</p> <p>Acoustic Tile</p> <p>Tube Spacer at Screw</p> <p><b>IFC-98 Track</b></p>	<p><b>Mounting to Plaster or Sheet Rock Ceiling</b></p> <p>2" x 3/16" Toggle Bolts 36" O.C.</p> <p>Plaster/Sheet Rock</p> <p><b>IFC-50 Track</b></p>
<p><b>Direct Mounting to Concrete Ceiling</b></p> <p>Lead or Plaster Anchor With #10 x 1" Pan Head Sheet Metal Screw 36" O.C.</p> <p><b>IFC-1000 IV Track</b></p>	<p><b>Mounting Thru Suspended Ceiling to Wood Blocking Above</b></p> <p>Ground Above Grid, May Be Notched, Wired to Tee or Left Free</p> <p>Acoustic Tile</p> <p>#10 x 2" Wood Screws 36" O.C.</p> <p><b>AE Screening</b></p>

## Installation

Imperial Cubicle and IV Tracks are designed to utilize a number of installation methods. Installation details are provided with all orders.

## Guarantee

Every Imperial Cubicle and IV Track is covered by an unconditional one year guarantee against defects in materials and workmanship.

You can depend on Imperial Fastener Company for exceptional value, outstanding service and superior quality Intravenous Support Systems, Cubicle Track, and Curtains (For more information on Cubicle Curtains, see our Sweet's File 10190/IMP).

When you're looking for the best, look to Imperial ...for Proven Performance!



833  
**JORGENSEN LABORATORIES**  
 1450 N. VAN BUREN AVE.  
 LOVELAND, CO. 80538  
 1-800-525-5614

33069

*1 5/8" wide*