

**T**he Safety Stick provides priceless protection when administering injection to fractious or vicious domestic or wild animals.



**Two different models:**

**Safety Stick** allows needle penetration, then administration of medication or drug. Most pole syringes do not.



**Safety Stick**

This syringe allows you to obtain proper needle stick, then push the dosage plunger on the operator end.

- Uses a standard Monoject 3cc syringe.
- Mechanism can not be damaged by even the most powerful animal bite.
- Available in 3 different lengths.
- Easy to inject through cage or a safe distance.

**Specifications:**

Catalog #	Length	Weight
J-482A	3 ft.	12 oz.
J-482B	4 ft.	15 oz.
J-482C	6 ft.	20 oz.

**Syringe recommended:** 3cc Disposable Monoject

**Construction:** Aircraft Aluminum alloy and stainless steel.

**Finish:** Black anodized satin finish.

**Using the Safety Stick is as easy as 1, 2, 3.**

To install, place Monoject 3cc syringe on receiver and snap into place with thumb or fingers. Syringe is held securely.

Device will allow easy removal of syringe when plunger is depressed.

