



NEW VetBedJet (ONLY) cage adaptor J1034ZC
Instruction Sheet

Step 1



Take pronged side and insert through front side of cage.

Step 2



Waved connector will go inside cage.
Line up two halves for next step.

Step 3



Turn waved side clockwise to tighten around cage bars.

Step 4



Cage adaptor will be secured on cage once tight.

Step 5



Connect VetBedJet hose to the outside connection.

Step 6



Place blanket of desired size inside cage and tie around waved side of connector.

Step 7



With the hose and blanket attached you are now ready to use your warm air unit.